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(Válida únicamente en los EE.UU.)

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IBMWEHC-C

Hot & Cold Therapy with the Power of Magnets: Elbow Wrap

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Hora del este
de lunes a viernes

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HoMedics, Inc.
3000 Pontiac Trail
Commerce Township, MI
48390
Impreso en China

Experience Hot & Cold Therapy with the Power of Magnets.

Thank you for purchasing this HoMedics TheraP product.

This product, like the entire TheraP line, is made with high quality craftsmanship to provide you with years of dependable service. We hope that you will find it to be the finest product of its kind.

Magnetic Wave™ Technology

WARNING

If you suffer from an injury or an ailment, consult a physician for proper medical advice. This product should never be used as a replacement for professional medical treatment.

Some individuals may be sensitive to the neoprene or adhesives used in this product. If soreness or rash occurs discontinue use immediately and consult your physician. Do not wear while sleeping. If problem persists, consult your physician.

Caution

- If at any time soreness, rash, or an increase in pain occurs, discontinue use and consult your physician.
- Only use this product for its intended purposes as described in the manual.
- Do not use this product over an open wound.
- Do not use this product while using transdermal drug delivery skin patches.
- Individuals with pacemakers should never use this product.
- Do not use during pregnancy.
- Children should not use or play with this product.
- Do not machine-wash magnets.
- Magnets should be kept at least six inches away from electronic devices, computers, TVs, video equipment and tapes.
- Keep out of direct contact with credit cards and other cards that have a magnetic strip. Should the magnet come in contact with your card, it may demagnetize and render your card invalid.

Hot & Cold Therapy

WARNING

Heated gel insert may cause serious burns if too hot. Do not use over sensitive skin or where there is poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.

Caution

- Do not use this product for more than 20 minutes at a time on the affected area.
- Consult a physician prior to use.
- If contents of gel insert come in contact with eyes, flush eyes with water. Contact a physician if irritation persists.
- Never apply gel insert directly to skin. If gel insert is uncomfortable during use, remove it immediately.
- Do not use gel insert with sport creams, balms, liniments or lotions.
- Children should not use or play with gel insert.
- Do not insert gel pack directly into microwave as this may cause damage to the seal.
- Use heat pack carefully, may cause severe burns.
- This product, while non-toxic, is not intended for ingestion. If contents are swallowed, do not induce vomiting. Consult a physician.
- Do not use hot/cold packs over skin areas with less than normal sensitivity.
- Do not use heated gel insert on parts of body where there is swelling, redness or immediately after an injury. Cold therapy is recommended for these conditions.
- Do not use gel insert over open wounds.

Washing

Remove gel insert before washing. Hand wash; line dry.

Instructions for Use

Cold Therapy

Place gel insert in freezer for a minimum of 2 hours to achieve optimal temperature and performance. Insert will remain flexible even when frozen. Remove from freezer and insert in pocket of elbow wrap.

Heat Therapy

Stovetop

1. Bring water to a boil and remove from heat.
2. Place gel insert in hot water for 1 minute. Do not boil insert directly.
3. Remove insert using tongs. Use caution as the insert will be very hot.
4. Test temperature of insert by placing it on the underside of your wrist. If insert is too hot, allow it to cool before using. If it is not hot enough, continue heating in 1 minute intervals until desired temperature is reached.
5. Place gel insert in pocket of elbow wrap.

Microwave

1. Heat water in microwaveable container on high setting for 3 minutes. This is based on a 600 watt microwave. Use a minimum of 16 ounces of water. Heating times may vary. Follow all precautions and recommendations found in your microwave oven instruction manual with respect to the heating of water. Do not insert gel pack directly into microwave as this may cause damage to the seal. After heating, let water stand in microwave for thirty seconds before moving it or inserting the gel pack.

- Place gel insert in hot water for 1 minute.
- Remove insert using tongs. Use caution as the insert will be very hot.
- Test temperature of insert by placing it on the underside of your wrist. If insert is too hot, allow it to cool before using. If it is not hot enough, continue heating in 1 minute intervals until desired temperature is reached.
- Place gel insert into pocket of wrap.

Fitting Instructions

- Position wrap over elbow. Inside of wrap should face skin.
- Secure fastening tabs above and below elbow, readjusting until comfortable. Do not secure wrap too tightly, as it may restrict blood circulation

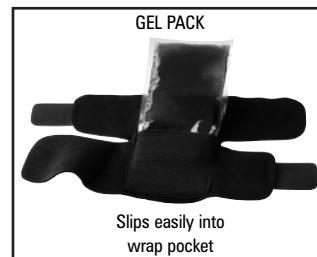
8 POWERFUL NEODYMIUM
(NDFEB) MAGNETS

DURABLE, QUALITY CONSTRUCTION
FOR YEARS OF DEPENDABLE USE.

PREMIUM QUALITY
NEOPRENE
Retains natural body heat
while helping to compress
the injured area.

CONVENIENT HOOK &
LOOP CLOSURES
For quick and easy
fastening.

COMFORTABLE, LIGHTWEIGHT DESIGN
Wear it right under your clothes.

ONE SIZE FITS MOST

90-Day Limited Warranty

(Valid in USA only)

HoMedics, Inc., guarantees this product free from defects in material and workmanship for a period of 90 days from the date of purchase, except as noted below.

This HoMedics product warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of HoMedics. This warranty is effective only if the product is purchased and operated in the USA. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under warranty. HoMedics shall not be responsible for any type of incidental, consequential or special damages. All implied warranties, including but not limited to those implied warranties of fitness and merchantability, are limited in the total duration of 90 days from the original purchase date.

To obtain warranty service on your Magnetic Wave Therapy product, either hand deliver or mail the unit and your dated sales receipt (as proof of purchase), postpaid, along with a check or money order in the amount of \$5.00 payable to HoMedics, Inc. to cover handling.

Mail To: HoMedics Service Center, Dept. 168, 3000 Pontiac Trail, Commerce Township, MI 48390

Upon receipt, HoMedics will repair or replace, as appropriate, your Magnetic Wave Therapy product and return it to you, postpaid. Warranty is solely through HoMedics Service Center. Service of this product by anyone other than HoMedics Service Center voids warranty.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state. Because of individual state regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: www.homedics.com

©2004-2010 HoMedics, Inc. and its affiliated companies, all rights reserved. HoMedics is a registered trademark of HoMedics, Inc. and its affiliated companies. TheraP™ and Magnetic Wave™ are trademarks of HoMedics Inc. and its affiliated companies. All rights reserved.

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MW-EHC

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For service or repair, do not return this unit to the retailer. Contact HoMedics Consumer Relations at:

Mail To:
HoMedics, Inc.
Service Center Dept. 168
3000 Pontiac Trail
Commerce Township, MI
48390

Email:
cservice@homedics.com

Phone:
1-800-466-3342
Business Hours:
8:30am-5 pm ET
Monday-Friday

Distributed by
HoMedics
HoMedics, Inc.
3000 Pontiac Trail
Commerce Township, MI
48390
Made in China

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