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IB-NMS600

# Back & Shoulder Percussion Massager with heat

Instruction Manual and  
Warranty Information

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**2** year  
limited warranty

## IMPORTANT SAFETY INSTRUCTIONS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING

### DANGER – TO REDUCE THE RISK OF ELECTRIC SHOCK:

- Always unplug the appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for an appliance that has fallen into water. Unplug it immediately.
- DO NOT use while bathing or in a shower.
- DO NOT place or store appliance where it can fall or be pulled into sink or tub.
- DO NOT place or drop in water or other liquid.

### WARNING – TO REDUCE THE RISK OF BURNS, ELECTRIC SHOCK, FIRE OR INJURY TO PERSONS:

- An appliance should NEVER be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- Not for use by children.
- Use this appliance only for its intended use as described in this manual. **DO NOT** use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to HoMedics Service Center for examination and repair.
- Keep cord away from heated surfaces.
- NEVER drop or insert any object into any opening.
- **DO NOT** operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Never cover the appliance when it is in operation. Excessive heating can occur and cause fire, electric shock or injury to persons.
- DO NOT carry this appliance by the power cord or use cord as handle.

**Caution:**  
**All servicing of this massager must be performed by authorized HoMedics service personnel only.**

- To disconnect, turn all controls to the “off” position, then remove plug from outlet.
- **DO NOT** use outdoors.
- NEVER operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.
- NEVER operate on a soft surface such as a bed or couch where the air openings may be blocked.
- Use heated surfaces carefully. May cause serious burns. **DO NOT** use over insensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.
- **DO NOT** use this massager in close proximity to loose clothing, jewelry, or long hair.

## SAVE THESE INSTRUCTIONS

**CAUTION—PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.**

**Note: There is a 15-minute auto shut off on the unit for your safety.**

- This product is not intended for medical use. It is intended only to provide a luxurious massage.
- Consult your doctor prior to using this product, if you are pregnant, you have a pacemaker or you have any concerns regarding your health.
- Not recommended for use by Diabetics.
- Never leave the appliance unattended, especially if children are present.
- **DO NOT** use this product for more than 15 minutes at a time.
- Extensive use could lead to the product’s excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- Never use this product directly on swollen or inflamed areas or skin eruptions.
- **DO NOT** use this product as a substitute for medical attention.
- The product is equipped with overheating protection circuit, if the power is cut-off, please wait until the product is cooled down.
- NEVER use this product while in bed.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user’s capacity to operate the controls.
- This unit should not be used by children or invalids without adult supervision.
- NEVER use this product in automobiles.
- Household use only.

## Instructions for Use

---

To reduce the risk of electric shock, this appliance is equipped with a polarized plug (one blade wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. **DO NOT** change the plug in any way.

1. Untie the cord strap, extend cord and insert plug into 120-volt household electrical outlet to power unit.
2. Place massager around neck or on area of back you wish to massage. See page 6 for instructions on the use of the hand control.

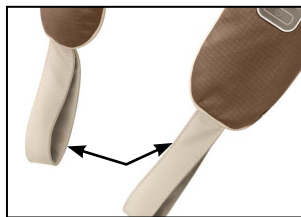
Fig. 1

Fig. 2

## Maintenance

### To Store

Place massager in its box or in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface. To avoid breakage, **DO NOT** wrap the power cord around the unit. **DO NOT** hang the unit by the power cord or flex handles.

### To Clean

Unplug the unit and allow it to cool before cleaning. Spot clean only with a soft, slightly damp cloth. Never allow water or any other liquids to come into contact with the unit.

- **DO NOT** immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, gasoline, kerosene, glass/furniture polish or paint thinner to clean.

- **DO NOT** attempt to repair the Back and Shoulder Percussion Massager. There are no user serviceable parts. For service, call the Consumer Relations telephone number listed in the Warranty Section.

## FCC Statement

---

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

**NOTE:** HoMedics is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user authority to operate the equipment.

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

### 1. Power Button

To turn on the massage functions, first press the POWER button. The L.E.D. indicator will illuminate and confirm your selection. The massager will begin in standard mode. To turn off the massage functions, simply press the button again.

### 2. Heat Button

For soothing heat, when massage is on, simply press the heat button and the corresponding red L.E.D. light will illuminate. To deselect, press the button again and the corresponding L.E.D. light will turn off. For your safety, heat cannot be turned on if no massage program is selected.

### 3. Demo Button

For a brief demonstration of the programs of the neck massager, press the Demo button. The L.E.D light will blink to indicate active function. The unit will briefly go through each program. Once completed, the unit automatically turns off.

### 4. Intensity Button

Choose the strength of your massage experience. Toggle through three massage intensities: Low, Medium and High. To select an intensity, simply press the button and the L.E.D. above the button will illuminate. To change the intensity, press the button again. Pressing the button once the highest intensity is reached will return it to its lowest setting.

### 5. Programs Button

The massager will start in standard mode. You can then choose from 3 preset programs where the massage rhythms change to loosen muscles in different ways. To select a program, simply press the button and the L.E.D above the button will illuminate. To select another program, simply press the button again. Pressing the button again after reaching the last program will return it to the first program.

# Back & Shoulder Percussion Massager with heat

### 1. Percussion Massage

Powerful action massage relaxes tense muscles

### 2. Flex Handles

Let's you move the massage nodes up and down your back

### 3. Convenient and Portable

Perfect for use at home or in the office

**Note:** The actual product materials and fabrics may vary from the picture shown.

**Note:**

As with all high-quality massage products, there is a 15-minute auto shut-off on the massaging mechanism.

Distributed by  
**HOMEDICS®**

**For service in the USA**

Email: [cservice@homedics.com](mailto:cservice@homedics.com)

Monday-Friday 8:30am-7:00pm est

1.800.466.3342

**For service in Canada**

Email: [cservice@homedicsgroup.ca](mailto:cservice@homedicsgroup.ca)

Monday-Friday 8:30am-5:00pm est

1.888.225.7378

**Limited Two Year Warranty**

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of two years from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your HoMedics product, contact a consumer relations representative for assistance. Please make sure to have the model number of the product available.

HoMedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics.

**This warranty is effective only if the product is purchased and operated in the country in which the product is purchased.** A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from country to country. Because of individual country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: [www.homedics.com](http://www.homedics.com).  
For Canada, please visit: [www.homedics.ca](http://www.homedics.ca).

HoMedics is a registered trademark of HoMedics, LLC.  
©2014 HoMedics, LLC. All rights reserved.  
IB-NMS600

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