# HOMEDICS™

#1 BRAND IN MASSAGE\*

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[www.homedics.com/register](http://www.homedics.com/register)

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## CORDLESS SHIATSUTALK

INSTRUCTION MANUAL AND WARRANTY INFORMATION

TWO YEAR LIMITED WARRANTY

NMS-675H

## IMPORTANT SAFETY INSTRUCTIONS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING

**DANGER** – TO REDUCE THE RISK OF ELECTRIC SHOCK:

- Always unplug the appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for an appliance that has fallen into water. Unplug it immediately.
- DO NOT use while bathing or in a shower.
- DO NOT place or store appliance where it can fall or be pulled into a tub or sink.
- DO NOT place in or drop into water or other liquid.
- Never use pins or other metallic fasteners with this appliance.

**WARNING** – TO REDUCE THE RISK OF BURNS, FIRE, ELECTRIC SHOCK, OR INJURY TO PERSONS:

- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on, or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to HoMedics Service Center for examination and repair.
- Keep the cord away from heated surfaces.
- NEVER drop or insert any object into any opening.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electric shock or injury to persons.
- DO NOT carry this appliance by the supply cord or use cord as a handle.
- To disconnect, turn all controls to the “off” position, then remove plug from outlet.
- DO NOT use outdoors.
- Never operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.
- Use heated surfaces carefully. May cause serious burns. Do not use over insensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.

- Do not use this massager in close proximity to loose clothing or jewelry.
- Keep long hair away from massager while in use.
- Never operate on a soft surface such as a bed or couch where the air openings may be blocked.
- **Caution: All servicing of this massager must be performed by authorized HoMedics service personnel only.**

## SAVE THESE INSTRUCTIONS

**CAUTION** – PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

- This product is not intended for medical use. It is intended only to provide a luxurious massage.
- Consult your doctor prior to using this product, if
  - You are pregnant
  - You have a pacemaker
  - You have any concerns regarding your health
- Not recommended for use by diabetics.
- Never leave the appliance unattended, especially if children are present.
- Do not use this product for more than 15 minutes at a time.
- Extensive use could lead to the product’s excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- Never use this product directly on swollen or inflamed areas or skin eruptions.
- DO NOT use this product as a substitute for medical attention.
- This product is equipped with an overheating protection circuit. If the power is cut off, please wait until the product is cooled down.
- NEVER use this product while in bed.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user’s capacity to operate the controls or who has sensory deficiencies.
- This product is not for use in automobiles.
- This product is intended for household use only.

## INSTRUCTIONS FOR USE

1. To charge, plug the adapter into a 120 volt AC outlet. The LED lights under the power button will flash in sequence as the product charges. When fully charged, all 3 LED lights will illuminate without flashing.
2. Place massager around neck (Fig. 1) or area of back you wish to massage. Convenient straps help assist in easy positioning and intensity control. On the next page you will find detailed instructions on how to operate the voice control feature of this product.

Fig. 1

Fig. 2

## VOICE CONTROL INSTRUCTIONS:

1. To utilize the voice control function of this massager, first press the power button. Your shiatsu massage will begin on the high speed setting. Then press the voice control icon .
2. Next, say "HoMedics". The product will respond "I am here". You may then speak any of the following commands, and the massager will respond with the desired action.

### SPEED ADJUSTMENT

To change the speed of your massage, first say "HoMedics" and wait for the product's response. The product will respond "I am here." Next say "medium speed" to change shiatsu speed to medium, or "low speed" to change speed to low. The product will confirm your selection by saying "ok, medium speed" or "ok, low speed".

### SOOTHING HEAT

If you wish to activate the soothing heat function of the massager, first say "HoMedics" and wait for the product to respond "I am here". Once the product responds, say "add heat". The product will respond "Ok, heat" and the heat function will activate. The Shiatsu nodes will glow red. To turn off the soothing heat, first say "HoMedics" and wait for the response. Once the product responds, say "heat off". The product will respond "Ok, heat off" and the heat function will deactivate.

### CHANGE DIRECTION

If you wish to reverse the direction of your shiatsu massage, first speak "HoMedics" and wait for the product to respond "I am here". Then, speak "change direction". The product will respond "Ok, change direction".

### STOP

When you are finished with your massage, say "HoMedics" and wait for the product to respond "I am here". Then, say "stop" and the product will respond "ok, stop". The Shiatsu massage will stop, and heat (if activated) will turn off.

To turn the massager on again, say "HoMedics, and wait for the products response. Then say, "Start". The product will respond "Ok, start", and your massage will begin again.

## MAINTENANCE

### TO CLEAN

Unplug the unit and allow it to cool before cleaning. Spot clean only with a soft, slightly damp sponge. Never allow water or any other liquids to come into contact with the unit.

- DO NOT immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, or strong chemicals that might or might not be flammable and/or damaging to the product to clean.
- DO NOT attempt to repair the massager. There are no user-serviceable parts. For service, contact the Consumer Relations telephone number listed in the warranty section.
- This product comes with a rechargeable lithium battery. This battery is not user serviceable. Please dispose in accordance with local, state, province, and country regulations.

### TO STORE

Place massager in its box or in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface. To avoid breakage, do not wrap the power cord around the unit. Do not hang the unit by the power cord.

### CONVENIENT AND PORTABLE

The rechargeable battery makes this product perfect for use at home or in the office.

**Note:** There is a 15-minute auto shutoff on the unit for your safety.

### VOICE CONTROL BUTTON

Press this button to activate the hands-free voice control function of this massager.

### SHIATSU REVERSE BUTTON

The massager will begin with an inward rotating motion. To change the direction of the massage nodes simply press the shiatsu reverse button and the nodes will begin rotating in an outward motion. Pressing the button again will change the direction back to an inward rotating direction.

### HEAT BUTTON

For soothing heat when massage is on, simply press the heat button and the massage nodes on the neck unit will illuminate. To deselect, press the button again and the corresponding massage nodes will stop glowing.

### POWER BUTTON

To turn on the massage functions, press the power button once. The massage will begin in high speed. Press the button a second time for medium speed, a third time for low speed and a fourth time to switch off the massager.

## FCC STATEMENT

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: HoMedics is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

## TWO YEAR LIMITED WARRANTY

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of two years from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your HoMedics product, contact a consumer relations representative for assistance. Please make sure to have the model number of the product available.

HoMedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state and country to country. Because of individual state and country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: [www.homedics.com](http://www.homedics.com). For Canada, please visit: [www.homedics.ca](http://www.homedics.ca).

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© 2017 HoMedics, LLC. Tous droits réservés. HoMedics est une marque déposée de HoMedics, LLC. HoMedics #1 Brand In Massage est une marque commerciale de HoMedics, LLC. IB-NMS675H