## ULTRA PREMIUM BACKREST SUPPORT

OFUF-BLK

English, Español

### Congratulations

on your purchase of the ObusForme by HoMedics UltraForme Backrest Support.

Your backrest support contains an “S”-shaped frame that supports your back for more comfortable sitting anywhere.

### Use and Care Tips:

#### Using your ObusForme by HoMedics UltraForme Backrest Support:

1. Place your backrest as far back as possible against the back of the seat or chair where you will sit.
2. Sit back into the center of the backrest. Adjust the height of the backrest so the curve fits most comfortably into your lumbar curve (small of your back).
3. Position and lock the straps on the backrest around the back of the seat/chair at the height established in Step 2. Tighten the straps by pulling on the ends.

**Please Note:** The UltraForme backrest support may take some time to get used to. If you feel discomfort when using your backrest for the first time, it is recommended that you use it for shorter periods of time, gradually increasing your usage until you feel comfortable.

#### Cleaning your UltraForme Backrest Support:

- Wash the cover of your backrest support with a soft cloth and water. DO NOT rub excessively.
- DO NOT machine wash

The UltraForme backrest support can be used with any ObusForme by HoMedics seat for a complete and comfortable ergonomic sitting experience!

### Limited One Year Warranty

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of one year from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your HoMedics product, mail the product and your dated sales receipt (as proof of purchase), postpaid, to the following address:

HoMedics Consumer Relations  
Service Center Dept. 168  
3000 Pontiac Trail  
Commerce Township, MI 48390

No COD's will be accepted.

HoMedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND

TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from country to country. Because of individual country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: [www.homedics.com](http://www.homedics.com)

**For service or repair, do not return this unit to the retailer. Contact HoMedics Consumer Relations at:**

**Mail:**  
HoMedics, Inc.  
Service Center Dept. 168  
3000 Pontiac Trail  
Commerce Township, MI  
48390

**Email:** [cservice@homedics.com](mailto:cservice@homedics.com)

**Phone:** 1-800-466-3342  
**Business Hours:** 8:30am-5pm ET  
Monday-Friday

**Distributed by**  
**HOMEDICS**<sup>®</sup>  
HoMedics, Inc.  
3000 Pontiac Trail  
Commerce Township, MI  
48390

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