**Therapist  
Select™**

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# Professional Percussion Massager

**Instruction Manual and  
Warranty Information**

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IB-PA1H-A

# Therapist Select™

## Designed by Professionals, for Professionals.

Thank you for purchasing HoMedics Therapist Select Ultra Percussion Massager. This, like all of HoMedics products, is built with high-quality craftsmanship and comes backed by our two-year limited warranty. Therapist Select Ultra Percussion Massager is guaranteed to provide you soothing massage relief for years to come.

HoMedics is dedicated to bringing you better health through innovation. However, you should never substitute this, or any other manufacturer's massager, for professional medical attention. Persistent ailments and symptoms could indicate a more serious condition.

## Professional Percussion Massager Features

- High intensity massage with therapeutic thermal heat
- Dual pivoting heads for an invigorating massage
- Variable speed control
- Lightweight, contoured design
- Ergonomic, foam cushioned handle
- 4 custom attachment sets included

## IMPORTANT SAFETY INSTRUCTIONS:

When using an electrical appliance, basic precautions should always be followed, including the following:

### READ ALL INSTRUCTIONS BEFORE USING

- DANGER:** To reduce the risk of electric shock:
- Always unplug the appliance from the electrical outlet immediately after using and before cleaning.
  - Do not reach for an appliance that has fallen into water. Unplug it immediately.
  - Do not use while bathing or showering.
  - Do not place or store appliance where it can fall or be pulled into a tub or sink. Do not place in or drop into water or other liquid.

2

## WARNING:

- To reduce the risk of burns, fire, electric shock or injury to persons:
- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.
  - Close supervision is necessary when this appliance is used by, or near children, invalids or disabled persons.
  - Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
  - Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to HoMedics Service Center for examination and repair.
  - Keep cord away from heated surfaces.
  - Never block the air openings of the appliance or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep air openings free of lint, hair, etc.
  - Never use while sleeping or fall asleep while using the massager.
  - Never drop or insert any object into any opening.
  - Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
  - Do not use on or near eyes or other highly sensitive areas.
  - Do not operate under blanket or pillow. Excessive heating can occur and cause fire, electrical shock or injury to persons.
  - Do not carry this appliance by supply cord or use cord as handle.
  - Do not use the vibrating massager directly on one concentrated area of the body. Vibrating massagers are designed to be used in a continuous motion, constantly changing the massage area.
  - To disconnect, turn all controls to the "OFF" position, then remove plug from outlet.
  - This appliance is designed for household use only.

3

## SAVE THESE INSTRUCTIONS

### Caution – Please read all instructions carefully before operating.

- If you have any concerns regarding your health consult your doctor before using this product.
- Individuals with pacemakers should consult a doctor before use.
- Never leave the appliance unattended, especially if children are present.
- Never cover the appliance when it is in operation.
- Do not use this product for more than 20 minutes at a time.
- Extensive use could lead to the products excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- Never use this product directly on swollen or inflamed areas or skin eruptions.
- This product is a non-professional appliance designed to provide soothing massage to worn muscles. Do not use this product as a substitute for medical attention.
- Do not use this product before bed. The massage has a stimulating effect and can delay sleep.
- Never use this product while in bed.
- This product should never be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls.
- This unit should not be used by children without adult supervision.

## Instructions for Use

- This appliance has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.
- 1. Always make sure the unit is set to the "OFF" position before inserting the plug into a 120 volt electrical outlet.
- 2. **Message:** To turn the unit on, simply slide the ON/OFF switch, located along the top of the unit's handle up one notch. To adjust the intensity of the massage, slide the selector switch down for lower intensity, up for higher intensity. (See page 7)
- 3. **Message with Thermal Heat:** Locate the two red (heat) attachments. Attach to the massage head by turning clockwise to secure. Slide the ON/OFF button two notches up for massage with infrared heat. Allow a few minutes for heat to fully activate. **NOTE: To eliminate the risk of being burned, never use the heat function without the attachments. Do not use the thermal heat continuously for more than 20 minutes. Do not use the thermal if you have sensitive skin, poor circulation, discolored areas, swelling, burned, inflamed or where skin eruptions or sores are present. Heat should be pleasant and comfortable. Should pain or discomfort result, discontinue use and consult your physician.**
- 4. Apply massager head lightly to the area you wish to treat.
- 5. Move the unit slowly, holding on the targeted area. Treat the area for a short period of time and move to another area. Never treat the same spot longer than 3 minutes. **CAUTION** - as stated in the "IMPORTANT SAFETY INSTRUCTIONS" section of this manual, never use this appliance on or near eyes or other highly sensitive areas.

- 6. Operation of longer than 20 minutes is not recommended and may cause overheating.
- 7. When treatment is completed, turn unit off, allow to cool, unplug the power supply and store properly in a dry place.

### To Change Attachments

Your massager comes equipped with four custom massage head attachments. Each set is a different density:

- Black** - Soft
- Medium grey** - Medium hardness
- Light grey** - Hard
- Red** - Heat

**Caution:**  
**All servicing of this unit must be performed by authorized HoMedics service personnel only.**

To change to a different set, simply twist off (counterclockwise) the set that is in the unit and twist on (clockwise) the desired set.

### Maintenance

**TO CLEAN:**

Be sure to unplug the unit and allow it to cool before cleaning. Use a soft, slightly damp cloth to wipe. Never allow water or any liquids to come into contact with the handle.

**TO STORE:**

Unplug the appliance from the outlet and allow it to cool before storing in its box or a clean, dry place. Coil the cord and secure it with the supplied fastener. Do not hang the unit by the powercord.

# Professional Percussion Massager

# Therapist Select™

## TWO YEAR LIMITED WARRANTY

(Valid in USA only)

HoMedics, Inc., guarantees this product free from defects in material and workmanship for a period of two years from the date of original purchase, except as noted below.

This HoMedics product warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of HoMedics. This warranty is effective only if the product is purchased and operated in the USA. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under warranty. HoMedics shall not be responsible for any type of incidental, consequential or special damages. All implied warranties, including but not limited to those implied warranties of fitness and merchantability, are limited in the total duration of two years from the original purchase date.

Mail To:  
Consumer Relations  
HoMedics  
Service Center Dept. 168  
3000 Pontiac Trail  
Commerce Township, MI 48390

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PA-1H

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For more information regarding our product line in the USA, please visit: [www.homedics.com](http://www.homedics.com)

©1999-2001 HoMedics, Inc. or its affiliated companies, all rights reserved.  
HoMedics® is a registered trademark of HoMedics Inc. and its affiliated companies.  
Therapist Select™ is a trademark of HoMedics Inc. and its affiliated companies. All rights reserved.

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